Geistlich Bio-Gide®

Geistlich Bio-Gide® Compressed

Geistlich Bio-Gide® Perio

Geistlich
Biomaterials

 **Manufacturer**
Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland
Tel.: +41 41 492 55 55
info@geistlich.com
www.geistlich-pharma.com

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CE CE Mark

Composition

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is a resorbable bilayer collagen membrane obtained by standardized, controlled manufacturing processes. The collagen is obtained from veterinary certified pigs and is carefully purified to minimize the risk of immunological reactions.

The membrane has a bilayer structure represented by a porous surface and a dense surface, and consists of native collagen without further cross-linking or chemical additives. The membrane is sterilized by γ -irradiation. STERILE R

Properties / Effects

The low immunogenicity and proven biocompatibility favor the use of Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio in oral and maxillofacial surgeries. Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio have smoothed surfaces. This feature prolongs the water uptake, facilitates application and cutting.

The membrane is used for hard and soft tissue regenerative procedures.

When used as a barrier membrane, the porous surface facing the bone allows for the ingrowth of bone-forming cells. The dense surface facing the soft tissue prevents the ingrowth of fibrous tissue into the defect.

The fibrous microstructure of the membrane is hydrophilic and readily absorbs fluid. The membrane retains its structural integrity when wet. When applied, the membrane adheres well to the surrounding and underlying tissues.

In vivo degradation of the membrane takes several months depending on the surgical site, vascularization and patient comorbidities. Due to the natural origin of the collagen, variations in the thickness of the membrane may be noted. Fixation with sutures or pins is possible. The membrane is not radio-opaque.

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Intended use

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is intended to be used in the treatment of hard and soft tissue defects in oral and maxillofacial surgery. The concerned hard and soft tissue defects may be surgically created or result from traumatic injuries, pathological conditions, medical treatments and therapies, or history of personal life-style habits.

Indications

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is indicated to be used in the surgical treatment of the following clinical conditions:

- > Tissue deficiencies in the alveolar ridge
- > Dentofacial anomalies and malformations.

Instructions for use

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio should only be used by licensed dentists/physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using the membrane.

- > The membrane may be trimmed to obtain the required size and shape. Geistlich Bio-Gide® Perio package contains sterile stencil templates in four different basic periodontal shapes.
- > When using Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio in bone regeneration procedures, the underlying bone deficiency can be filled with a space-maintaining material (e.g., Geistlich Bio-Oss®), if necessary. In such cases, the membrane should overlap the walls of the defect by at least 2 mm to allow complete bone contact and to prevent connective tissue ingrowth lateral to the material.
- > The dense surface of the membrane (Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed: marked with «UP» at the membrane margin) should face the soft tissue, and the porous surface should face the defect.
- > To achieve adhesion, the membrane should be applied over the prepared site and held carefully in place to enable penetration of the membrane by blood and oral fluids. This allows good adaptation and adhesion of the membrane to the surrounding surfaces and facilitates the formation of a blood clot.
- > Fixation of the membrane is possible due to its high tensile strength and can be applied, if necessary.
- > The membrane can be left exposed for tension-free closure.

Special instructions for use in periodontology

- > A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention, patients must undergo an extensive hygiene phase. A postoperative maintenance phase is important for a long-term therapeutic success.
- > To avoid the formation of a long junctional epithelium, Geistlich Bio-Gide® Perio must be adapted closely to the tooth (if necessary, with additional fixation using suture material).

Post-operative care

The usual post-operative care, medication and instructions should be provided. The grafted area should not be brushed for the first four weeks and excessive muscle tractioning or trauma to the treated areas should be avoided for the first four weeks.

Limitation of use / Precautions

Contraindications

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio must not be used:

- > in the presence of acutely infected wounds
- > in patients with a known allergy to collagen.

Precautions

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio should be used with special caution in patients with:

- > chronic infection (e.g., periodontitis) at the surgical site
- > uncontrolled metabolic diseases (e.g., diabetes, osteomalacia, thyroid disorder)
- > prolonged corticosteroid therapy
- > autoimmune diseases
- > radiotherapy
- > heavy smoking
- > any medications and clinical conditions that negatively influence tissue healing.

Geistlich Bio-Gide® Perio: **Do not implant the stencil templates!** Their perforations enable them to be distinguished from the membrane.

Intraoperatively, if there is need to remove Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the membrane is integrated over time with the existing tissue.

The membrane is intended for use in a single patient in a single medical procedure. Any membrane not required in the single procedure should be discarded since sterility and stability cannot be assured once the original primary packaging is opened.

Local use of the membrane with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

The membrane is a typical non-metallic diamagnetic material, in common with most human tissues. It cannot be heated or act as a magnet during the MRI examination. The membrane has not been specifically studied in the MR environment. The membrane may be used with metal hardware. In this case, special caution under the MR environment may be required.

Side effects

As Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is a collagen product, in very rare cases allergic reactions may occur. Possible side effects that may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflam-

mation/necrosis, and infection. These side effects could lead to reduced tissue healing. Other surgery-related disturbances that may appear include bone loss and pain.


Pregnancy / Lactation

Data on the use of Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio during pregnancy or lactation are not available.

Other information

Do not re-use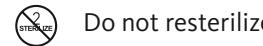


Do not resterilize

Use by date

Temperature limit

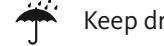Keep dry


Do not use if the package is damaged


Keep away from sunlight

Caution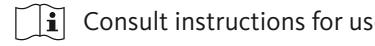

Consult instructions for use

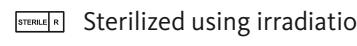Sterilized using irradiation

Batch code

Catalogue number

Recycling symbol

Authorized representative in the European Community / European Union

Importer

Sterile Barrier

Prior to use, the integrity of the sterile barrier system must be visually inspected for breaches. Do not use the product if packaging is damaged or opened unintentionally.

Presentation

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is packed in sterile double-blister packaging:

Geistlich Bio-Gide®, 1 membrane, size 13 mm x 25 mm

Geistlich Bio-Gide®, 1 membrane, size 25 mm x 25 mm

Geistlich Bio-Gide®, 1 membrane, size 30 mm x 40 mm

Geistlich Bio-Gide® Compressed, 1 membrane, size 13 mm x 25 mm

Geistlich Bio-Gide® Compressed, 1 membrane, size 20 mm x 30 mm

Geistlich Bio-Gide® Perio, 1 membrane, size 16 mm x 22 mm + sterile stencil templates;

Perio-System Combi-Pack

1 Geistlich Bio-Gide® Perio, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg) + sterile templates

Geistlich Combi-Kit Collagen

1 Geistlich Bio-Gide®, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg)

Patient Implant Card (PIC)

All information in regard of the provided PIC can be accessed under the following URL:

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