Geistlich Bio-Gide® Shape

Geistlich
Biomaterials

Manufacturer

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CE Mark

Composition

Geistlich Bio-Gide® Shape is a pre-shaped resorbable bilayer collagen membrane obtained by standardized, controlled manufacturing processes. The collagen is obtained from veterinary certified pigs and is carefully purified to minimize the risk of immunological reactions.

The membrane has a bilayer structure represented by a porous surface and a dense surface, and consists of native collagen without further cross-linking or chemical additives. The membrane is sterilized by γ-irradiation. 

Properties / Effects

The low immunogenicity and proven biocompatibility favor the use of Geistlich Bio-Gide® Shape in oral and maxillofacial surgeries.

Geistlich Bio-Gide® Shape has smoothened surfaces. This feature prolongs the water uptake, facilitates application and cutting.

The membrane is used for hard and soft tissue regenerative procedures.

When used as a barrier membrane, the porous surface facing the bone allows for the ingrowth of bone-forming cells. The dense surface facing the soft tissue prevents the ingrowth of fibrous tissue into the defect.

The fibrous microstructure of the membrane is hydrophilic and readily absorbs fluid. The membrane retains its structural integrity when wet.

In vivo degradation of the membrane takes several months depending on the surgical site, vascularization and patient comorbidities. Due to the natural origin of the collagen, variations in the thickness of the membrane may be noted. Fixation with sutures or pins is possible. The membrane is not radio-opaque.

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Intended use

Geistlich Bio-Gide® Shape is intended to be used in the treatment of hard and soft tissue defects in oral and maxillofacial surgery.

The concerned hard and soft tissue defects may be surgically created or result from traumatic injuries, pathological conditions, medical treatments and therapies, or history of personal life-style habits.

Indications

Geistlich Bio-Gide® Shape is designed to be used for the treatment of alveolar bone defects in extraction sockets.

Geistlich Bio-Gide® Shape can also be used in the surgical treatment of the following clinical conditions:

- > Tissue deficiencies in the alveolar ridge
- > Dentofacial anomalies and malformations.

Instructions for use

Geistlich Bio-Gide® Shape should only be used by licensed dentists/physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using the membrane.

When used in the treatment of defective extraction sockets.

- > The tooth is extracted and the usual surgical procedures (curettage, de-epithelization, etc.) are undertaken.
- > The long strap of the Geistlich Bio-Gide® Shape should be placed to cover the bony defect of the socket, if present. Thereby, the dense surface of the membrane should face the soft tissue, and the porous side should face the defect.
- > Place the membrane without pre-wetting within the extraction socket and held in place with moderate pressure. Adhesion to the bone surface is facilitated by the hydrophilic properties of the membrane.
- > The defect is filled with a space-maintaining material (e.g., Geistlich Bio-Oss®, Geistlich Bio-Oss® Collagen). The membrane should overlap the walls of the defect to prevent connective tissue ingrowth.
- > Complete penetration of the membrane by blood and oral fluids facilitates adaptation and adhesion of the membrane to the defect and the formation of a blood clot.
- > To close the extraction socket, both lateral wings of the Geistlich Bio-Gide® Shape membrane should be tucked underneath the mucosa, located mesially and distally of the socket. The upper wing should be tucked lingually and palatally in-between the soft tissue and bone wall.
- > Fixation of Geistlich Bio-Gide® Shape (e.g., by mattress suture) may be indicated to avoid displacement of the membrane and the space-maintaining material.

Post-operative care

The usual post-operative care, medication and instructions should be provided. The grafted area should not be brushed for the first four weeks and excessive muscle tractioning or trauma to the treated areas should be avoided for the first four weeks.

Limitation of use / Precautions

Contraindications

Geistlich Bio-Gide® Shape must not be used:

- > in the presence of acutely infected wounds
- > in patients with a known allergy to collagen.

Precautions

Geistlich Bio-Gide® Shape should be used with special caution in patients with:

- > chronic infection (e.g., periodontitis) at the surgical site
- > uncontrolled metabolic diseases (e.g., diabetes, osteomalacia, thyroid disorder)
- > prolonged corticosteroid therapy
- > autoimmune diseases
- > radiotherapy
- > heavy smoking
- > any medications and clinical conditions that negatively influence tissue healing.

Intraoperatively, if there is a need to remove Geistlich Bio-Gide® Shape, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the membrane is integrated over time with the existing tissue.

The membrane is intended for use in a single patient in a single medical procedure. Any membrane not required in the single procedure should be discarded since sterility and stability cannot be assured once the original primary packaging is opened.

Local use of the membrane with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

The membrane is a typical non-metallic diamagnetic material, in common with most human tissues. It cannot be heated or act as a magnet during the MRI examination. The membrane has not been specifically studied in the MR environment. The membrane may be used with metal hardware. In this case, special caution under the MR environment may be required.

Side effects

As Geistlich Bio-Gide® Shape is a collagen product, in very rare cases allergic reactions may occur. Possible side effects that may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation/necrosis, and infection. These side effects could lead to reduced tissue healing. Other surgery-related disturbances that may appear include bone loss and pain.

Pregnancy / Lactation

Data on the use of Geistlich Bio-Gide® Shape during pregnancy or lactation are not available.

Other information

Do not re-use

Do not re-sterilize

Use by date

Temperature limit

Keep dry

Do not use if the package is damaged

Keep away from sunlight

Caution

Consult instructions for use

Sterilized using irradiation

Batch code

Catalogue number

Recycling symbol

Authorized representative in the European Community / European Union

Importer

Sterile Barrier

Prior to use, the integrity of the sterile barrier system must be visually inspected for breaches. Do not use the product if packaging is damaged or opened unintentionally.

Presentation

Geistlich Bio-Gide® Shape is packed in a sterile double-blister pack:

Geistlich Bio-Gide® Shape, 1 membrane, pre-shaped, 14 x 24 mm

Patient Implant Card (PIC)

All information in regard of the provided PIC can be accessed under the following URL:
pic.geistlich-pharma.com

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Geistlich Pharma AG