Geistlich Bio-Oss® Collagen

Geistlich
Biomaterials

Manufacturer

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CE Mark

Composition

Geistlich Bio-Oss® Collagen is a mixture of 90 wt-% Geistlich Bio-Oss® (granules of the size 0.25 – 1 mm) and 10 wt-% porcine collagen fibers in a block form. Geistlich Bio-Oss® is a natural bone mineral of bovine origin. Because of its natural origin, Geistlich Bio-Oss® is chemically as well as structurally comparable to the human bone mineral (nanocrystalline natural apatite). Sterilization of Geistlich Bio-Oss® Collagen is carried out by γ -irradiation. 

Properties / Effects

The spongy consistency of the material allows simple trimming. The collagen facilitates adaptation of the Geistlich Bio-Oss® to the defect site. The collagen is resorbed within several weeks. Geistlich Bio-Oss® supports the ingrowth of new bone at the implantation site by serving as a stable scaffold for osteoblasts; over time it is subject to the physiological remodeling process. The mineral phase is slowly resorbable, depending on the physiological environment of the individual patient, or integrated into the host bone.

The highly purified osteoconductive mineral structure is produced from natural bone in a multistage purification process, adhering to the strictest safety regulations. Due to its properties, Geistlich Bio-Oss® Collagen is a proven alternative to autologous bone when used for bone regeneration in oral-maxillofacial areas. Geistlich Bio-Oss® Collagen is radiopaque like any phosphocalcic bone substitute.

Intended Use

Geistlich Bio-Oss® Collagen is intended to be used in the treatment of hard tissue defects in oral-maxillofacial surgery.

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Indications

Geistlich Bio-Oss® Collagen is indicated in the following clinical situations:

- > bone deficiencies in the alveolar ridge
- > dentofacial anomalies and malformations.

Instructions for use

- > Geistlich Bio-Oss® Collagen is to be used by licensed dentists / physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss® Collagen.
- > Complete elimination of granulation tissue following exposure of the defect prior to the application of the device.
- > Geistlich Bio-Oss® Collagen may be cut to the required size with sterile tweezers and scissors, in dry form or after moistening with blood or sterile, non-pyrogenic physiological saline solution. The material is applied using sterile instruments.
- > In situ modeling may be performed with a sterile spatula or other suitable instrument.
- > To minimize the risk of wound dehiscence or displacement of Geistlich Bio-Oss® Collagen, it is advisable to stabilize or cover the treated defect (e.g., with Geistlich Bio-Gide® or Geistlich Mucograft® Seal).

Special instructions for use in periodontology

A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention, patients must receive a hygiene phase treatment, consisting of oral hygiene instructions, scaling and root planing, and occlusal adjustment when indicated.

Besides plaque control, the filling of periodontal defects with Geistlich Bio-Oss® Collagen requires successful local treatment of the periodontal lesion (root planing, debridement) prior to the implantation.

Post-operative care

The usual post-operative care, medication and instructions should be given to patients.

Limitation of use / Precautions

Contraindications

- > Geistlich Bio-Oss® Collagen must not be used in the presence of acutely infected wounds.
- > Geistlich Bio-Oss® Collagen must not be used in patients with a known allergy to collagen.

Precautions

Geistlich Bio-Oss® Collagen should be used with special caution in patients with:

- > chronic infection (e.g. osteomyelitis) at the surgical site
- > uncontrolled metabolic disorders, e.g. diabetes, osteomalacia, thyroid disorder, severe renal or hepatic disorder
- > prolonged corticosteroid therapy
- > immunological impairment diseases
- > radiotherapy
- > heavy smoking

> or any medications that negatively influence bone healing.

To support sufficient regeneration and integration of bone, Geistlich Bio-Oss® Collagen should be implanted only into vital bony tissue and in direct contact with the host bone (if necessary with microfracture of the bone surface). For larger defects, the addition of autologous spon-
gious bone or bone marrow may improve the regeneration process.

Mechanical loading and insertion of implants (two-stage procedure) in the augmented area should be avoided until several weeks after the application of Geistlich Bio-Oss® Collagen. The appropriate timing of implant insertion depends on the residual local bone volume.

Intraoperatively, if there is need to remove the product, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the product is intended to resorb over time and will be integrated into the host bone.

Local use of Geistlich Bio-Oss® Collagen with medicinal products, alcohol, disinfectants or an-
tibiotics has not been studied.

Geistlich Bio-Oss® Collagen is a typical non-metallic diamagnetic material, like most human tissues. It cannot be heated or act as a magnet during the MRI examination. Geistlich Bio-Oss® Collagen has not been specifically studied in the MR environment.

The product is intended for use in a single patient in a single medical procedure. Any product not required in the single procedure should be discarded, since sterility and stability cannot be assured once the original primary packaging is opened.

Side effects

Incompatibility reactions with Geistlich Bio-Oss® Collagen cannot be totally excluded. As Geistlich Bio-Oss® Collagen contains collagen, in very rare cases allergic reactions may occur. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection, transient discomfort, or pain.

Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation and in children before skeletal maturity are not available.

Other Information

Do not re-use

Do not re-sterilize

Use-by date

Temperature limit

Keep dry

Do not use if package is damaged

Keep away from sunlight

Caution

Consult electronic instructions for use

Sterilized using irradiation

Batch code

Catalogue number

Recycling symbol

Authorized representative in the European Community / European Union

Importer

Sterile Barrier

Prior to use, the integrity of the sterile barrier system must be visually inspected for breaches. Do not use the product if packaging is damaged or opened unintentionally.

Presentation

Geistlich Bio-Oss® Collagen: double-blister pack with one block

Block sizes:

> 50 mg (approx. 2.5 mm x 5.0 mm x 7.5 mm)

> 100 mg (approx. 5.0 mm x 5.0 mm x 7.0 mm)

> 250 mg (approx. 7.0 mm x 7.0 mm x 7.0 mm)

> 500 mg (approx. 10.0 mm x 10.0 mm x 7.0 mm)

Perio-System Combi-Pack:

1 Geistlich Bio-Gide® Perio, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg) + sterile templates

Geistlich Combi-Kit Collagen:

1 Geistlich Bio-Gide®, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg)

Patient Implant Card (PIC)

Information in regard of the provided PIC can be accessed under the following URL:
pic.geistlich-pharma.com

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Geistlich Pharma AG