# Geistlich Perio-System Combi-Pack

**Geistlich**  
Biomaterials

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CE Mark

## Composition

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is a resorbable bilayer collagen membrane obtained by standardized, controlled manufacturing processes. The collagen is obtained from veterinary certified pigs and is carefully purified to minimize the risk of immunological reactions.

The membrane has a bilayer structure represented by a porous surface and a dense surface, and consists of native collagen without further cross-linking or chemical additives. The membrane is sterilized by  $\gamma$ -irradiation. 

## Properties / Effects

The low immunogenicity and proven biocompatibility favor the use of Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio in oral and maxillofacial surgeries. Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio have smoothed surfaces. This feature prolongs the water uptake, facilitates application and cutting.

The membrane is used for hard and soft tissue regenerative procedures.

When used as a barrier membrane, the porous surface facing the bone allows for the ingrowth of bone-forming cells. The dense surface facing the soft tissue prevents the ingrowth of fibrous tissue into the defect.

The fibrous microstructure of the membrane is hydrophilic and readily absorbs fluid. The membrane retains its structural integrity when wet. When applied, the membrane adheres well to the surrounding and underlying tissues.

In vivo degradation of the membrane takes several months depending on the surgical site, vascularization and patient comorbidities. Due to the natural origin of the collagen, variations in the thickness of the membrane may be noted. Fixation with sutures or pins is possible. The membrane is not radio-opaque.

# Geistlich

Biomaterials

 **Manufacturer**  
Geistlich Pharma AG  
Bahnhofstrasse 40  
6110 Wolhusen  
Switzerland  
Tel.: +41 41 492 55 55  
info@geistlich.com  
www.geistlich-pharma.com

## Intended use

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is intended to be used in the treatment of hard and soft tissue defects in oral and maxillofacial surgery. The concerned hard and soft tissue defects may be surgically created or result from traumatic injuries, pathological conditions, medical treatments and therapies, or history of personal life-style habits.

## Indications

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is indicated to be used in the surgical treatment of the following clinical conditions:

- > Tissue deficiencies in the alveolar ridge
- > Dentofacial anomalies and malformations.

## Instructions for use

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio should only be used by licensed dentists/physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using the membrane.

- > The membrane may be trimmed to obtain the required size and shape. Geistlich Bio-Gide® Perio package contains sterile stencil templates in four different basic periodontal shapes.
- > When using Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio in bone regeneration procedures, the underlying bone deficiency can be filled with a space-maintaining material (e.g., Geistlich Bio-Oss®), if necessary. In such cases, the membrane should overlap the walls of the defect by at least 2 mm to allow complete bone contact and to prevent connective tissue ingrowth lateral to the material.
- > The dense surface of the membrane (Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed: marked with «UP» at the membrane margin) should face the soft tissue, and the porous surface should face the defect.
- > To achieve adhesion, the membrane should be applied over the prepared site and held carefully in place to enable penetration of the membrane by blood and oral fluids. This allows good adaptation and adhesion of the membrane to the surrounding surfaces and facilitates the formation of a blood clot.
- > Fixation of the membrane is possible due to its high tensile strength and can be applied, if necessary.
- > The membrane can be left exposed for tension-free closure.

## Special instructions for use in periodontology

- > A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention, patients must undergo an extensive hygiene phase. A postoperative maintenance phase is important for a long-term therapeutic success.
- > To avoid the formation of a long junctional epithelium, Geistlich Bio-Gide® Perio must be adapted closely to the tooth (if necessary, with additional fixation using suture material).

## Post-operative care

The usual post-operative care, medication and instructions should be provided. The grafted area should not be brushed for the first four weeks and excessive muscle tractioning or trauma to the treated areas should be avoided for the first four weeks.

## Limitation of use / Precautions

### Contraindications

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio must not be used:

- > in the presence of acutely infected wounds
- > in patients with a known allergy to collagen.

### Precautions

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio should be used with special caution in patients with:

- > chronic infection (e.g., periodontitis) at the surgical site
- > uncontrolled metabolic diseases (e.g., diabetes, osteomalacia, thyroid disorder)
- > prolonged corticosteroid therapy
- > autoimmune diseases
- > radiotherapy
- > heavy smoking
- > any medications and clinical conditions that negatively influence tissue healing.

Geistlich Bio-Gide® Perio: **Do not implant the stencil templates!** Their perforations enable them to be distinguished from the membrane.

Intraoperatively, if there is need to remove Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the membrane is integrated over time with the existing tissue.

The membrane is intended for use in a single patient in a single medical procedure. Any membrane not required in the single procedure should be discarded since sterility and stability cannot be assured once the original primary packaging is opened.

Local use of the membrane with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

The membrane is a typical non-metallic diamagnetic material, in common with most human tissues. It cannot be heated or act as a magnet during the MRI examination. The membrane has not been specifically studied in the MR environment. The membrane may be used with metal hardware. In this case, special caution under the MR environment may be required.

## Side effects

As Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is a collagen product, in very rare cases allergic reactions may occur. Possible side effects that may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflam-

mation/necrosis, and infection. These side effects could lead to reduced tissue healing. Other surgery-related disturbances that may appear include bone loss and pain.

## Pregnancy / Lactation

Data on the use of Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio during pregnancy or lactation are not available.

## Other information

Do not re-use

Do not re-sterilize

Use by date

Temperature limit

Keep dry

Do not use if the package is damaged

Keep away from sunlight

Caution

Consult instructions for use

Sterilized using irradiation

Batch code

Catalogue number

Recycling symbol

Authorized representative in the European Community / European Union

Importer

Sterile Barrier

Prior to use, the integrity of the sterile barrier system must be visually inspected for breaches. Do not use the product if packaging is damaged or opened unintentionally.

## **Presentation**

Geistlich Bio-Gide® / Geistlich Bio-Gide® Compressed / Geistlich Bio-Gide® Perio is packed in sterile double-blister packaging:

Geistlich Bio-Gide®, 1 membrane, size 13 mm x 25 mm

Geistlich Bio-Gide®, 1 membrane, size 25 mm x 25 mm

Geistlich Bio-Gide®, 1 membrane, size 30 mm x 40 mm

Geistlich Bio-Gide® Compressed, 1 membrane, size 13 mm x 25 mm

Geistlich Bio-Gide® Compressed, 1 membrane, size 20 mm x 30 mm

Geistlich Bio-Gide® Perio, 1 membrane, size 16 mm x 22 mm + sterile stencil templates;

Perio-System Combi-Pack

1 Geistlich Bio-Gide® Perio, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg) + sterile templates

Geistlich Combi-Kit Collagen

1 Geistlich Bio-Gide®, 16 mm x 22 mm + 1 Geistlich Bio-Oss® Collagen (100 mg)

## **Patient Implant Card (PIC)**

All information in regard of the provided PIC can be accessed under the following URL:

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CE Mark

## Composition

Geistlich Bio-Oss® Collagen is a mixture of 90 wt-% Geistlich Bio-Oss® (granules of the size 0.25 – 1 mm) and 10 wt-% porcine collagen fibers in a block form. Geistlich Bio-Oss® is a natural bone mineral of bovine origin. Because of its natural origin, Geistlich Bio-Oss® is chemically as well as structurally comparable to the human bone mineral (nanocrystalline natural apatite). Sterilization of Geistlich Bio-Oss® Collagen is carried out by  $\gamma$ -irradiation. 

## Properties / Effects

The spongy consistency of the material allows simple trimming. The collagen facilitates adaptation of the Geistlich Bio-Oss® to the defect site. The collagen is resorbed within several weeks. Geistlich Bio-Oss® supports the ingrowth of new bone at the implantation site by serving as a stable scaffold for osteoblasts; over time it is subject to the physiological remodeling process. The mineral phase is slowly resorbable, depending on the physiological environment of the individual patient, or integrated into the host bone.

The highly purified osteoconductive mineral structure is produced from natural bone in a multistage purification process, adhering to the strictest safety regulations. Due to its properties, Geistlich Bio-Oss® Collagen is a proven alternative to autologous bone when used for bone regeneration in oral-maxillofacial areas. Geistlich Bio-Oss® Collagen is radiopaque like any phosphocalcic bone substitute.

## Intended Use

Geistlich Bio-Oss® Collagen is intended to be used in the treatment of hard tissue defects in oral-maxillofacial surgery.

# Geistlich

Biomaterials

**Manufacturer**  
Geistlich Pharma AG  
Bahnhofstrasse 40  
6110 Wolhusen  
Switzerland  
Tel.: +41 41 492 55 55  
info@geistlich.com  
www.geistlich-pharma.com

## Indications

Geistlich Bio-Oss® Collagen is indicated in the following clinical situations:

- > bone deficiencies in the alveolar ridge
- > dentofacial anomalies and malformations.

## Instructions for use

- > Geistlich Bio-Oss® Collagen is to be used by licensed dentists / physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss® Collagen.
- > Complete elimination of granulation tissue following exposure of the defect prior to the application of the device.
- > Geistlich Bio-Oss® Collagen may be cut to the required size with sterile tweezers and scissors, in dry form or after moistening with blood or sterile, non-pyrogenic physiological saline solution. The material is applied using sterile instruments.
- > In situ modeling may be performed with a sterile spatula or other suitable instrument.
- > To minimize the risk of wound dehiscence or displacement of Geistlich Bio-Oss® Collagen, it is advisable to stabilize or cover the treated defect (e.g., with Geistlich Bio-Gide® or Geistlich Mucograft® Seal).

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A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention, patients must receive a hygiene phase treatment, consisting of oral hygiene instructions, scaling and root planing, and occlusal adjustment when indicated.

Besides plaque control, the filling of periodontal defects with Geistlich Bio-Oss® Collagen requires successful local treatment of the periodontal lesion (root planing, debridement) prior to the implantation.

## Post-operative care

The usual post-operative care, medication and instructions should be given to patients.

## Limitation of use / Precautions

### Contraindications

- > Geistlich Bio-Oss® Collagen must not be used in the presence of acutely infected wounds.
- > Geistlich Bio-Oss® Collagen must not be used in patients with a known allergy to collagen.

### Precautions

Geistlich Bio-Oss® Collagen should be used with special caution in patients with:

- > chronic infection (e.g. osteomyelitis) at the surgical site
- > uncontrolled metabolic disorders, e.g. diabetes, osteomalacia, thyroid disorder, severe renal or hepatic disorder
- > prolonged corticosteroid therapy
- > immunological impairment diseases
- > radiotherapy
- > heavy smoking

> or any medications that negatively influence bone healing.

To support sufficient regeneration and integration of bone, Geistlich Bio-Oss® Collagen should be implanted only into vital bony tissue and in direct contact with the host bone (if necessary with microfracture of the bone surface). For larger defects, the addition of autologous spongy bone or bone marrow may improve the regeneration process.

Mechanical loading and insertion of implants (two-stage procedure) in the augmented area should be avoided until several weeks after the application of Geistlich Bio-Oss® Collagen. The appropriate timing of implant insertion depends on the residual local bone volume.

Intraoperatively, if there is need to remove the product, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the product is intended to resorb over time and will be integrated into the host bone.

Local use of Geistlich Bio-Oss® Collagen with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

Geistlich Bio-Oss® Collagen is a typical non-metallic diamagnetic material, like most human tissues. It cannot be heated or act as a magnet during the MRI examination. Geistlich Bio-Oss® Collagen has not been specifically studied in the MR environment.

The product is intended for use in a single patient in a single medical procedure. Any product not required in the single procedure should be discarded, since sterility and stability cannot be assured once the original primary packaging is opened.

### Side effects

Incompatibility reactions with Geistlich Bio-Oss® Collagen cannot be totally excluded. As Geistlich Bio-Oss® Collagen contains collagen, in very rare cases allergic reactions may occur. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection, transient discomfort, or pain.

### Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation and in children before skeletal maturity are not available.

### Other Information

 Do not re-use

 Do not re-sterilize

 Use-by date

 Temperature limit

 Keep dry

 Do not use if package is damaged

 Keep away from sunlight

 Caution

 Consult electronic instructions for use

 Sterilized using irradiation

 Batch code

 Catalogue number

 Recycling symbol

 Authorized representative in the European Community / European Union

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